

MODJAW™ USER, ZAINSTALUJ OPROGRAMOWANIE TWIM ABY ROZPOCZĄĆ SWOJĄ PODRÓŻ 4D



WYKONAJ NASTĘPUJĄCE KROKI

- 1 – INSTALACJA
lub
- 2- AKTUALIZACJA
lub
- 3- DEZINSTALACJA



1 INSTALACJA

DOWNLOAD MODJAW SOFTWARE

MODJAW

LICENSE FILE

1- Pobierz

2 - Pobierz

INSTALLATION GUIDE

3- Uruchomienie



DOWNLOAD MODJAW

Download the installer and the license file

license.txt



ModJaw_installer_...exe
131/250 Mo, 10 s restantes

Langue de l'assistant d'installation



Veuillez sélectionner la langue qui sera utilisée par l'assistant d'installation.

English

4 - wybór języka

OK

Annuler

5 - kliknij

Setup - ModJaw



Welcome to the ModJaw Setup Wizard

This will install ModJaw version X.X.XXXXXXXX on your computer.

It is recommended that you close all other applications before continuing.

Click Next to continue, or Cancel to exit Setup.

6 - Kliknij

Next

Cancel

Setup - ModJaw

Select Destination Location

Where should ModJaw be installed?



Setup will install ModJaw into the following folder.

To continue, click Next. If you would like to select a different folder, click Browse.

C:\ModJaw

Browse...

At least 853,5 MB of free disk space is required.

7 - Kliknij

Back

Next

Cancel

Setup - ModJaw

Select Additional Tasks

Which additional tasks should be performed?



Select the additional tasks you would like Setup to perform while installing ModJaw, then click Next.

Additional shortcuts:

☒ Create a desktop shortcut

8 - Kliknij

9 - Kliknij

Back

Next

Cancel

1 INSTALACJA

Setup - ModJaw

Ready to Install

Setup is now ready to begin installing ModJaw on your computer.

Click Install to continue with the installation, or click Back if you want to review or change any settings.

Destination location:
C:\ModJaw

Additional tasks:
Additional shortcuts:
Create a desktop shortcut

10 - Kliknij

Back

Install

Cancel

Setup - ModJaw



Completing the ModJaw Setup Wizard

To complete the installation of ModJaw, Setup must restart your computer. Would you like to restart now?

☒ Yes, restart the computer now

☐ No, I will restart the computer later

11- Restart

Finish




12 - Uruchomienie oprogramowania

Hello
Sign in to ModJaw Twim



MODJAW

© 2022 ModJaw - All rights reserved

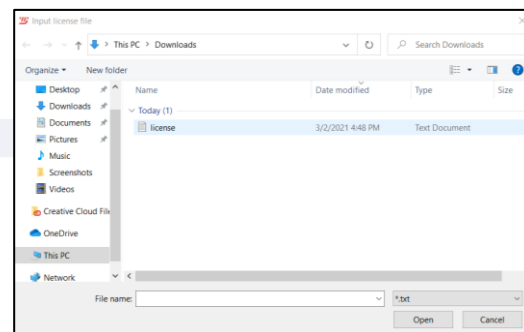

13- Wybierz plik licencji

SELECT LICENSE


ACTIVATE

14- Aktywuj

GRATULACJE, OPROGRAMOWANIE TWIM JEST AKTYWNE!

AKTUALIZACJA OPROGRAMOWANIA TWIM

Jeśli podczas uruchamiania oprogramowania TWIM zostanie wyświetlone następujące powiadomienie, oznacza to, że dostępna jest aktualizacja oprogramowania TWIM.

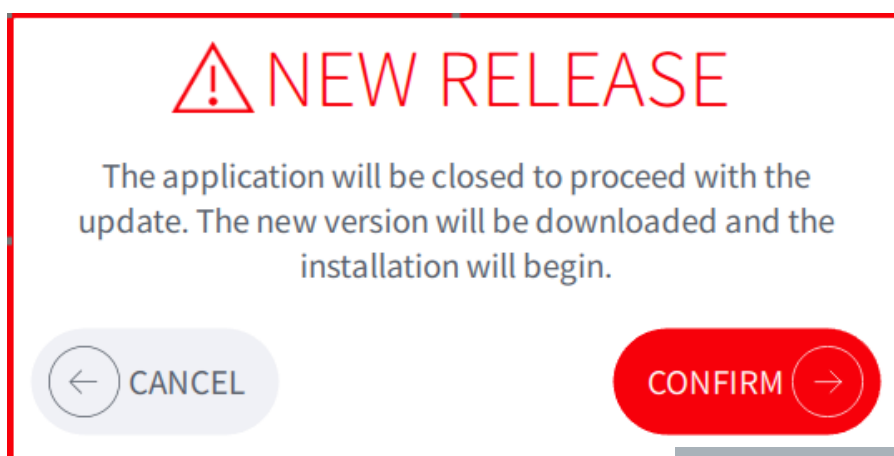


1- Kliknij i postępuj zgodnie z instrukcjami instalacji zawartymi w tym przewodniku

Deinstalacja TWIM... :-)

Przejdź do folderu instalacyjnego MODJAW:

 unins000.exe

1- Uruchomienie

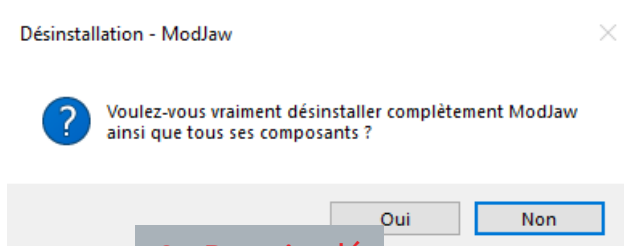

Uwaga:

- Dane i dzienniki nie są usuwane. Jeśli chcesz usunąć wszystkie dane Modjaw, usuń folder instalacji Modjaw
- Jeśli oprogramowanie jest już zainstalowane na stacji sprzętowej dostarczonej przez MODJAW (TIM™, Tech In Motion™), odinstalowanie oprogramowania MODJAW jest zabronione.

Jeśli masz jakiegokolwiek pytania lub problemy napotkane podczas procesu instalacji lub deinstalacji, skontaktuj się z pomocą techniczną MODJAW: support@modjaw.com.